

# MEDIDOR DE PEZONES

## INSTRUCCIONES DE IMPRESIÓN

1. Imprime esta guía de tallas al 100%. Comprueba la escala alineando el extremo de tu tarjeta de crédito con la línea de la parte inferior de la página.
2. Dobra los círculos por la mitad siguiendo la línea de puntos.
3. Recorta cada círculo con cuidado.

## SELECCIONA TU TALLA


1. Estimula tu pezón hasta que se reactive
2. Desliza los círculos sobre cada pezón.
3. 3. Selecciona el círculo que pueda deslizarse hasta la base sin tocar ninguna parte del pezón.
4. Calcula tu talla utilizando la tabla de medidas.

NOTA: El tallaje es un "arte". En función de la elasticidad de tus pezones, puede que necesites una talla inferior si tienes pezones elásticos, o una talla superior si son menos elásticos.

Si sientes molestias o experimentas una reducción constante del volumen, puede que necesites experimentar con diferentes tallas.

## SIZING CHART

Measured Nipple Size	Recommended flange size
up to 11 mm	15 mm
up to 14 mm	18 mm
up to 17 mm	21 mm
up to 20 mm	24 mm
up to 25 mm	27 mm
Our all-mom Customer Care team is here to help:	

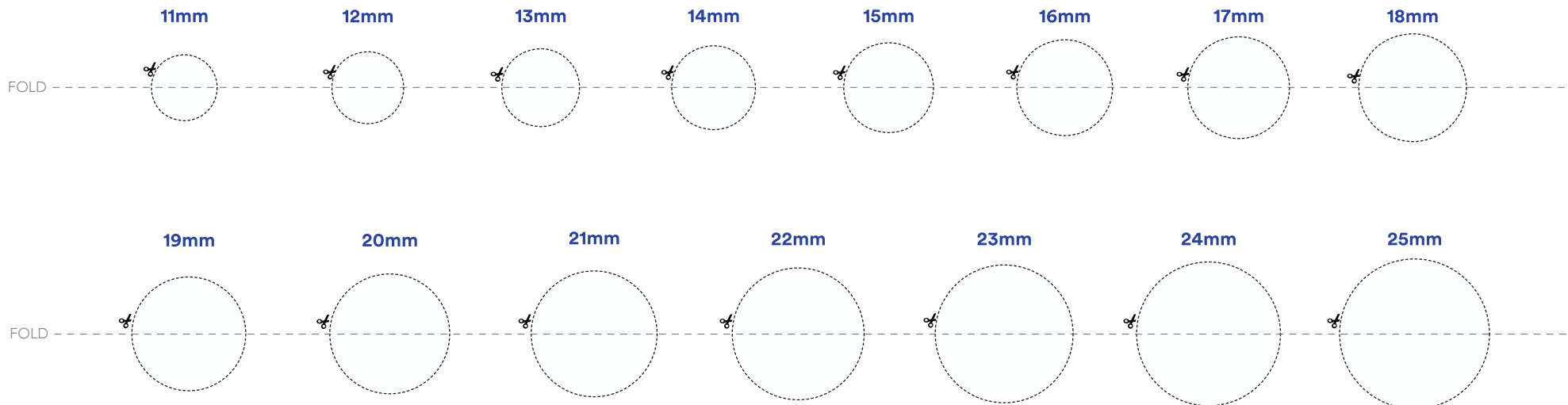

Note: Confirm scale is 100% by measuring the short end of a credit card to this line. If it doesn't match, reprint.

